roeko Stay-put

NON-IMPREGNATED

Instructions for use	EN
----------------------	----

Use:

A product for improving the results of dental impres-sions.

Application:

Cut *Stay-put* to the required length. *Stay-put* can be soaked in GINGIVA LIQUID or another gingival retraction solution of your choice.

Please note: Don't use GINGIVA LIQUID in cases of necrotizing Gingivitis ulcerosa. Please follow instructions for use of the chosen liquid. GINGIVA LIQUID is not available in all countries.

Important:

For best results when using *Stay-put* please follow in-structions carefully.

Place the centre of the cord into the sulcus. Next work out towards the ends of the cord.

Overlapping is not necessary.

Leave the non-impregnated cord for at least 5 min-utes in the sulcus. When using GINGIVA LIQUID allow 1-3 minutes for the liquid to take effect. Remove the cord directly before taking the impression.

Compositions:

Stay-put is a unique combination of softly braided re-traction cord and an ultra fine copper filament bounded with nylon.

0	x-fine	REF 521 000
1	fine	REF 521 001
2	medium	REF 521 002
3	thick	REF 521 003

Contraindications:

None

Further advise:

Avoid touching the cord with electrotomy and laser equipment.

Warning:

Medical devices should be kept out of reach of chil-dren.

Use and storage:

- for single use only
- for dental use only
- the lid has to be closed immediately after use and to be stored in a dry place
- shelf life 5 years
- this medical device should not be used after the ex-piration date

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Coltene/Whaledent GmbH + Co. KG
Coltene/Whaledent Inc.